





Follow these steps when applying


# FET Tags

- Apply with Zee Tags FET tagger
- Correct tag application ensures better tag retention


1. Place the tag into the FET tagger. The male part of the tag should be positioned in the yellow jaw of the tagger, with the female part positioned in the white jaw.

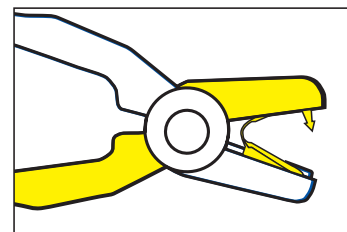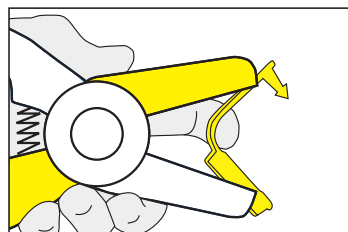

2. Dip the tagger holding the ear tag into antiseptic or disinfectant solution.

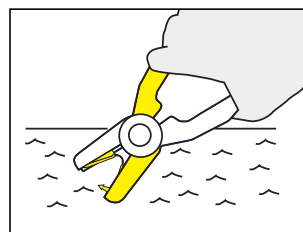

3. Apply the tag through the ear at 1/4 of the distance from the head avoiding the main blood vessels and the ridges of cartilage. In case of young animals remember to allow for ear growth by leaving the tag to stick out by 1/4 of its length from the ear.

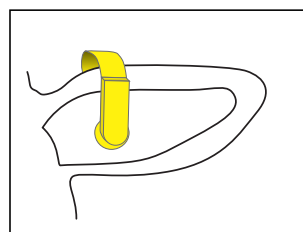

**Holding the animal securely and flattening the animal's ear while applying the tag makes the tagging process easier and more accurate.**

4. Always make sure the male part of the tag is fitted in the back of the ear. After applying the tag, the female part of the tag should be on the inside of the ear.

